


Over the Shoulder Harness

Part No. Y8103-OTSH


Installation

Our new Over the Shoulder Harness is an added way to give extra upper body support when being transported in a school bus.

Unbuckle existing vehicle seat belt, lift bench seat and insert the tongue part of the buckle end between seat cushion and backrest, leaving the 1" webbing (crotch strap) lay on the seat cushion. Lay the two looped ends over the top on the seat back. Buckle the unit on the backside of the seat back.

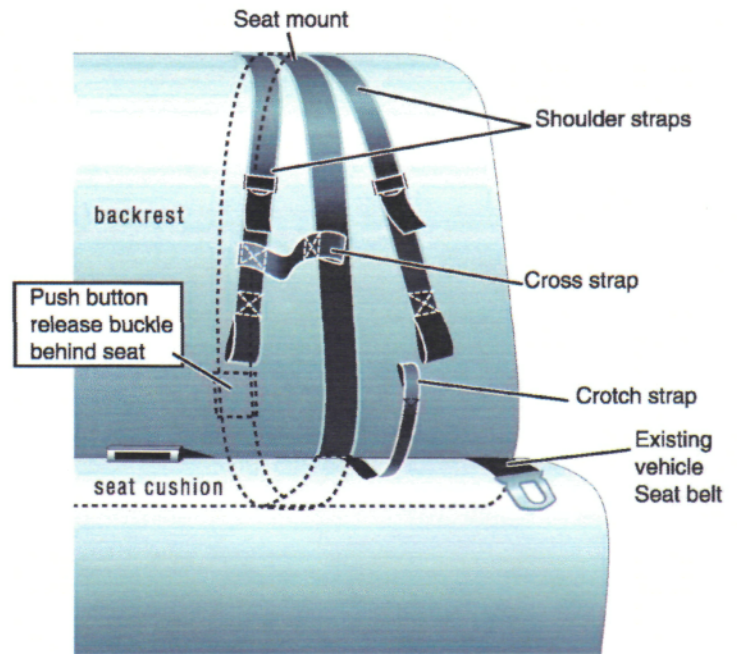

Place the passenger in the seat; the crotch strap should be between the legs. Thread the lap belt through the first loop then through the crotch strap loop, then through the remaining loop, buckle unit. Pull adjusting strap on seat back snug, and affix excess strap end.

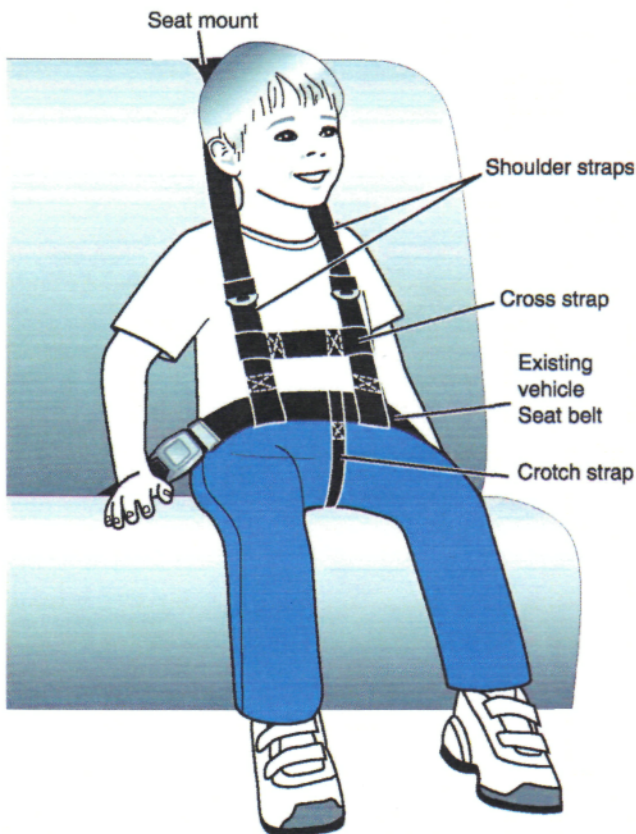

BESI, Inc.

9087 Sutton Place
Hamilton, Ohio 45011
Toll Free: 800-543-8222
Fax: 513-874-0579

- | | |
|-------------------------------------|-------------------------------------|
| <input type="checkbox"/> Y8103-1AHB | <input type="checkbox"/> Y8103-1CHB |
| <input type="checkbox"/> Y8103-1ALB | <input type="checkbox"/> Y8103-1CLB |

Inspected By: _____



Supplier Council Member